# SPORT-ELEC®

## ELECTROSTIMULATION

*Mode d'emploi*

User manual / Bedienungsanleitung / Manual de uso / Manuale d'uso / Gebruikshandleiding / Manual de utilização /  
Užívateľská príručka / Használati utasítás / Instrukcja obsługi / Manual de utilizare / Ъководство / Bruksanvisning /  
Руководство пользователя / Kullanım kılavuzu / دليل المستخدم

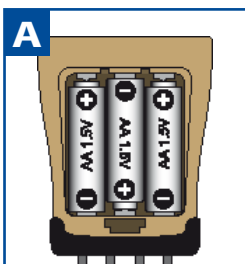
body control  
*System* **ABDOS / DUO**

[www.sport-elec.com](http://www.sport-elec.com)

I

II

**x 3**  
**AA LR6**  
**1.5 V**

- NON FOURNIES
- NOT SUPPLIED
- NICHT MITGELIEFERT
- NO INCLUIDAS
- NON FORNITE
- NIET GELEVERD
- NÃO FORNECIDAS
- NENÍ SOUČÁSTÍ
- NEM TARTOZÉK
- NIE DOSTARCZONE
- NU SUNT FURNIZATE
- КОИТО НЕ СА ДОСТАВЕНИ
- MEDFÖLJER INTE
- НЕ ПОСТАВЛЯЕТСЯ
- BIRLIKTE VERILMEZ
- غير مرفق

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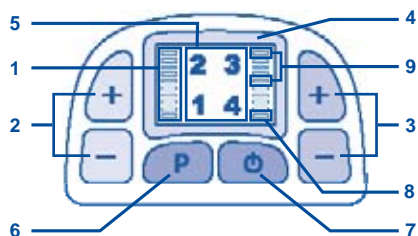
### CONFORMITÉ

Appareil ..... EN60601-1 / EN60601-2-10

## DESCRIPTION OF THE UNIT BODY CONTROL SYSTEM

- 1 - Indicator of power level
- 2 - Power key (Output A)  
+ : increase power / level 0 to 64  
- : decrease power / level 64 to 0
- 3 - Power key (Output B)  
+ : increase power / level 0 to 64  
- : decrease power / level 64 to 0
- 4 - LCD Display
- 5 - Program indicator
- 6 - Program selection key
- 7 - ON/OFF key
- 8 - Disconnection warning light:  
blinks when the belt (or the panty) is disconnected.
- 9 - Battery charge warning light :  
the 6<sup>th</sup> segment blinks : batteries are low  
the 10<sup>th</sup> segment blinks : batteries are used

## PROGRAMS

### PROG 1 FIRMNESS

Pulse rate = 70Hz  
ON time = 2.5 seconds  
OFF time = 2.5 seconds  
Ramp Up and Down = 2 seconds  
Program timer = 20 minutes

### PROG 2 MUSCLE CARE

Pulse rate = 60 Hz  
ON time = 3.5 seconds  
OFF time = 3.5 seconds  
Ramp Up and Down = 2 seconds  
Program timer = 26 minutes

### PROG 3 MUSCLE FORCE

Pulse rate = 50 Hz  
ON time = 4.5 seconds  
OFF time = 4.5 seconds  
Ramp Up and Down = 2 seconds  
Program timer = 30 minutes

### PROG 4 MASSAGE

Pulse rate = 50 Hz  
ON time = 5.5 seconds  
OFF time = 5.5 seconds  
Ramp Up and Down = 2 seconds  
Program timer = 40 minutes

## “ACHIEVING AN ADEQUATE MUSCLE CONTRACTION”

The most suitable effect is reached when you detect that the sensitivity of your muscle responds to the stimulation coming from the BODY CONTROL SYSTEM device, so that :

- You can feel the contraction
- You can see the contraction
- The contraction is palpable

In any case you must not feel a discomfort

**CUSTOMER SUPPORT: 00 33 2 32 96 50 50**

## INTRODUCTION

You have recently acquired the electro-musculature device with accessories, designed, produced and manufactured by SPORT-ELEC® INSTITUT (France). We thank you for your purchase and suggest that you read attentively the following paragraphs for an optimal use of the device. SPORT-ELEC® is specifically designed to enhance the abdominal area and the buttocks.

### GENERALITIES

BODY CONTROL SYSTEM (BCS) is conceived for women and men who care about their appearance. BCS is a device which uses electrical stimulation through the "contraction-release" effect on muscles that you want to work. The aim of the BCS is to refine your silhouette by acting on the muscular masses (waist, Abs, butt muscles, thighs, and legs). BCS contains 4 programs for a more particular development of certain muscles (pectoral, abdominal, biceps...) resulting from SPORT-ELEC® research. The frequencies and impulses width of each program have been defined to meet perfectly everybody's requirements :

- Work the muscle gently and facilitate its recovery after an effort
- Improve the firmness of tissues
- Reshape the silhouette

In conclusion, according to our activity, some muscles are more or less developed, the others atrophy on the contrary. Our body contains more than 600 muscles. The shape and harmony of our body depends on these muscles.

BCS allows activating the set of muscles without any effort from you. Its daily use will restore vigour of your muscle structure.

### THERAPEUTIC APPLICATIONS

- **Private:** to strengthen and / or to maintain muscle strength.
- **Physiotherapists:** to ease muscular tensions.
- **In hospital environment:** to prevent muscular atrophy and to treat pain.

### WARNING

READ THE FOLLOWING INFORMATION BEFORE USE :

The generator is provided with a safety system to avoid incorrect handling of the device by users.

Prior to use, check the following points :

- The device will stop if you try to start a program without positioning the belt on the body on first.
- Apply the contact cream to the electrodes (supplied with the device).
- The device must be fully connected to all the 4 connecting plugs of the accessory.
- Adjust accessories carefully to ensure contact between electrodes and the skin.

**CAUTION:** If you move with accessories attached and a partial loss of contact between electrodes and the skin occurs, the device will stop (safety stop).

### CONTRA-INDICATIONS

**Do not use SPORT-ELEC® :**

- \* On the heart area.
- \* If you carry a pacemaker or suffer from any other heart problem.
- \* If you have any form of an abdominal implant. The safety has not been established for the use of electronic muscle stimulators over abdominal implants.
- \* If you are pregnant.
- \* During first 6 weeks after baby birth.
- \* During first 3 months after having a surgery (like caesarean section...) and consult your doctor before using the device.
- \* If you have cancer, as the effects of electronic stimulation of cancerous tissue are unknown.
- \* In case of neuromuscular diseases.
- \* In case of haemorrhagic diseases.

## I. INSERTION OF BATTERIES

Check that the device is switched off.

Open the "batteries" compartment, insert three 1,5 V AA LR6 alkaline batteries into the compartment then close the case.

Comply with + and - indications (photo II - A page 2).

Do not mix used and good batteries.

For an optimal use of your device, it is advised to use good quality batteries.

## II. STARTING THE UNIT

Your electro-stimulation device is an appliance at the state-of-the-art technically, its design is based on standards EN 60601-1 and -2-10, it is foreseen for improving tone of your muscles, and not for measurement or diagnosis". The interface "man machine" is simplified and realized by means of numbers and function keys. From 1 to 4, the number in black indicates you the program chosen by pressing the "P" key.

Two graduated scales (1 by output) indicate you 64 power levels.

You also find 6 order keys (scheme, page 10) :

- 1 ON/OFF "O" - a short press
- 1 "P" to select the programs - a short press
- 2 "+/-" keys per output, in each side of the LCD display, to adjust the intensity of the stimulation.
- A long (1 sec.) press on the "\*" key allows to return the power level rapidly to zero.

#### FUNCTION 1 - "ON / OFF" SWITCH

- A short press on the "O" key puts the device under or off power.
- Every time under power, the device makes an auto-test on 11 essential parameters at the level of the safety of its operation. During the auto-test segments and pictograms are lit during 0,5 second.
- If an internal defect in the device is detected, all the pictograms flash. Send back the device to the After Sale Service.
- The auto-test does not take into consideration the used batteries. When the batteries are used the autotest cannot be performed.
- One long press (over 3 sec) on the "O" key starts up the auto-test.  
In this case, stop the device by pressing the "O" key and then press shortly the "O" key again.

#### FUNCTION 2 - CHOICE OF A PROGRAM "P"

Just make one or several presses on the "P" key, the programs ravel.

The programs are announced by numbers from 1 to 4.

Once the program is chosen, the program number is shown on the display. (see page 10).

Once the program is chosen, you can start it by increasing only one of the both "+/-" keys.

During the program the "P" key is inactive.

#### FUNCTION 3- END OF PROGRAM - STOP

- At the end of the program the device stops automatically.
- To stop the device during a program, you just have to turn 2 power keys to zero or to press the "O" key.
- In case of emergency, press the "O" key.

#### FUNCTION 4 - ADJUSTMENT THE POWER

You have 64 levels from 0 to 64 to adjust the electro-stimulation power.

Between two segments, there are 7 power levels or 4 short press on "+/-" key.

#### FUNCTION 5 - BATTERIES

- If the batteries are "good", there is no blinking.
- If the batteries are "weak", two segments (the 6th segment) blink (scheme page 10). Your device still works.
- If the batteries are "used", two segments (the 10th segment) blink (scheme page 10).
- If your device stops, even during a program: change the batteries.

#### FUNCTION 6 - DISCONNECTION

Your device is endowed with an accessory disconnecting detector.

If there is no contact between the skin and the accessory, the power is stopped on the concerned output (channel A or B).

You are warned of the disconnection by blinking of the 1<sup>st</sup> segment during 30 sec., after 30 sec. the blinking stops (diagram, page 10).

If no accessory is in contact with the body, the power cannot be emitted.

### III. CONTACT CREAM

The SPORT-ELEC® cream is used with accessory.

It also may be used for a manual massage of the body as well.

The cream facilitates the contact between electrodes and skin.

Its massaging and electroconducting properties improve comfort and efficiency of the stimulation.

If you wish to buy again some cream, order it directly at 02 32 96 50 50, on our website [www.sport-elec.co.uk](http://www.sport-elec.co.uk), or contact your local retailer.

#### WARNING

- You must not use the belt without having applied the cream on the built-in electrodes.
- Do not use any other cream except the one supplied by SPORT ELEC® Institut

#### SAFETY PRECAUTIONS BY USING THE CREAM

- In case of allergy, stop to apply the cream.
- Do not swallow.
- Do not leave with the range children.
- In case of contact with eyes rinse with water.

### IV. USE THE BOBY CONTROL SYSTEM WITH THE BELT

#### PRESENTATION OF THE BELT :

- **Front side (photo I - B page 2):** 1 = Wire storage 2 = Connection plugs 3 = Scratch strap
- **Electrode side (photo I - D page 2):** 4 built-in electrodes

#### STEP 1 : INSTALL AND CONNECT THE BELT

- Make sure that your BODY CONTROL SYSTEM unit is switched OFF.
- Take the connectors of the unit and plug them to the belt plugs by 2 groups (photo I - F page 2).
- Apply a thick layer of the electro-conductive massage cream on the built-in electrodes of the belt (the convex part - photo I - E page 2)
- Place the belt on your abdominal muscles in such a way that the electrodes should be centred on your abs. Adjust them using the "scratch" straps making sure that the built-in electrodes are in contact with the skin (photos I page 2 : B - abdominal muscles).

#### STEP 2 : START A SESSION

Once the unit is connected, make yourself comfortable and start your device following the procedure "Starting the unit" detailed previously in paragraph II (page 11).

Do not try to increase the intensity too quickly.

If the power seems being too strong, decrease it by pressing briefly on the "-." key.

A long press on the "-." key returns the power to its minimal level. This operation does not switch off the device.

Do not hesitate to place the belt slightly to the left or right (after switching the power key on zero) in order to feel the contractions on the whole abdominal area.

If an electrode hurts or if you feel some tingling, it is necessary to stop the device and to verify whether the electrode is well covered with cream.

**Caution :** Press the "+" key only during stimulation. Don't press the "+" key when there's no stimulation.

#### STEP 3 : END OF THE SESSION

Once the program is finished, switch off the device and unplug it from the belt.

Remove the surplus of cream from the built-in electrodes with a moistened cloth and dry the belt.

Store carefully your material.

### V. USE THE BOBY CONTROL SYSTEM WITH THE PANTY (BCS-DUO Only)

#### PRESENTATION OF THE PANTY :

- **Front side (photo I - G page 2):** 1 = Wire storage, 2 = Connection plugs, 3 = Scratch strap
- **Electrode side (photo I - D page 2):** 4 built-in electrodes

#### STEP 1 : CONNECT AND INSTALL THE PANTY

- Make sure that your SPORT-ELEC® device is switched OFF.
- Take the connectors of the unit and plug them to the panty plugs by 2 groups (photo I - F page 2).
- Apply a thick layer of the electro-conducting cream to the built-in electrodes of the panty (the convex part - photo I - E page 2)
- Place the panty on your butts (photos I page 2 : G = butts).

#### STEP 2 : START A SESSION

Once your device is connected, make yourself comfortable and start your device following the procedure "Starting the unit" detailed previously in paragraph II.

Do not try to increase the intensity too quickly.

If the power seems to you too strong, decrease it by pressing briefly on the "-." key.

A long press on the "-." key returns the power to its minimal level. This operation does not switch off the device.

Do not hesitate to place the panty slightly to the left or right (after switching the power key on zero) in order to feel the contractions on the whole area of buttocks.

If an electrode hurts or if you feel some tingling, it is necessary to stop the device and to verify whether the electrode is well covered with cream.

#### STEP 3 : END OF THE SESSION

Once the program is finished, switch off the device and unplug it from the panty.

Remove the surplus of cream from the built-in electrodes with a moistened cloth and dry the belt. Store carefully your material.

## VI. RECOMMENDATION FOR USE

### A / PRECAUTIONS

- Some persons may experience skin irritation or hypersensitivity due to the electrical stimulation or contact cream.
- Keep away from children or people with mental disabilities. The device contains small pieces that can be swallowed.
- Use this device only with the leads, electrodes, and accessories recommended by the manufacturer.
- Do not use portable electrical stimulators while driving, operating machinery, or during any activity in which electrical stimulation can put the user at risk of injury.

#### Use with caution in the following circumstances :

- persons with suspected or diagnosed epilepsy
- when there is a tendency to bleed internally following an injury of fracture
- following recent surgical procedures when stimulation may disrupt the healing process
- over areas of skin that lack normal sensation

#### GET YOUR DOCTOR'S ADVICE IF :

- You have back troubles. You should also ensure the intensity is kept low.
- You have any serious illness or injury not mentioned in this guide.
- You have recently undergone a surgical procedure.
- You take insulin for diabetes.

#### TIPS EXPLANATIONS FOR SMALL SIZES :

If the belt overlaps with electrodes on the body, fold the overlapping part of the belt by means of scratch sections.

#### TROUBLE SHOOTING

If your device doesn't work properly, do not use it and contact your retailer.

#### CONTACT SPORT-ELEC® INSTITUT OR AN AUTHORIZED PROVIDER IF :

- ▶ You experience any irritation, skin reaction, hypersensitivity or other adverse reaction. However, please note that some reddening of the skin is quite normal under the belt during and for a short time after exercise.
- ▶ An effective treatment should not cause undue discomfort. SPORT-ELEC® INSTITUT will not accept responsibility if the guidelines and instructions supplied with this unit are not followed.
- ▶ If you are in any doubt about using the device for any reason, please consult your doctor before use.

#### RECOMMENDATIONS OF USE

- You can restart a program's cycle on another muscle zone.
- For optimum use, refer to the diagrams shown in this manual.
- Clean the unit and its cables with a cloth dampened with a disinfecting solution diluted in water at 1/10<sup>th</sup>.
- If the device is not likely to be used for some time, the batteries should be removed

#### ⚠ RESTRICTIONS OF USE ⚠

##### To benefit fully of your electrostimulation session, please pay attention to the following guidelines :

- The belt must not be connected to other objects.
- Do not touch inside the belt while the BODY CONTROL SYSTEM is switched on.
- Do not use while driving or operating machinery.
- Do not use in wet environment.
- Use this device in a clean place (without dust nor dirt.).
- Keep your unit at distance from heat source.
- Keep your unit at distance from short-wave emitting equipment or high frequency surgical devices.
- To reposition the belt during a session :  
Always stop the program currently running, unfasten the belt and refasten it after the belt's position has been adjusted.
- After strenuous exercise or exertion :  
Always use a lower intensity to avoid muscle fatigue.

### B / WARNINGS

The long-term effects of electrical stimulation are unknown.

Wait before using your BODY CONTROL SYSTEM:

- When applying the belt, always remember to place the belt only on the abdomen and as indicated in this manual.
  - Do not apply stimulation on the neck. Severe spasm of the muscles may occur and the contractions may be strong enough to close the airway or cause difficulty in breathing.
  - Stimulation over the neck could also have adverse effects on the heart rhythm or blood pressure.
  - Do not apply stimulation across the chest because the introduction of electrical current into the chest may cause rhythm disturbances to the heart, which could be lethal.
  - The effects of stimulation on the brain are unknown. Therefore, do not apply stimulation across the head and do not place electrodes on the opposite side of the head.
  - Apply stimulation only to normal, intact, clean skin. Do not apply stimulation over open wounds or over swollen, infected, or inflamed areas or skin eruptions, e.g. phlebitis, thrombophlebitis, varicose veins, etc.
  - Do not apply stimulation in the bath or under the shower.
  - Do not use this device while sleeping.
  - If you have an IUD contraceptive device, do not practice electro-stimulation.
- The safety of use of this device when used with an IUD has not been established

#### WARNING :

- **Use only on those parts of the body for which the accessories were designed.**
- **Functions only on batteries**
- **Not to connect to another source of power**

#### ADVERSE REACTIONS

Skin irritation and burns beneath the electrodes have been reported with the use of electrical stimulators applied to the skin.

#### RECOMMENDATIONS

Please apply regulations enforced in your country when disposing your product at its end of life. Therefore, please discard it in a location planned to this effect, ensuring it is disposed in an environment-friendly manner

#### CLEANING AND MAINTENANCE CONDITIONS FOR THE BELT.

- No washing
- No solvent
- No drying
- No ironing
- No laundering
- Clean accessories with a humid sponge soaked with detergent and wipe with a cloth.
- Clean the unit with a cloth soaked in with disinfectant (Mercrlyl type) diluted with water at 1/10<sup>th</sup>.
- Clean the electrodes with a mix of water/hypoallergenic detergent, then rinse thoroughly.

## VIII. TECHNICAL SPECIFICATIONS

### UNIT :

Class IIa medical device in accordance with Directive 93/42/CEE	
Software version.....	1
Type of current.....	Dual phase rectangular and symetric
Number of programmes.....	4 programs
Number of adjustable independent outputs.....	2
Current.....	From 0 to 60mA from 0 to 60Vpp- from 0 to 1000 Ω
Frequency range / Pulse width range.....	50 - 70 Hz / 200 μs
Power.....	3x 1,5 V alkaline batteries type LR6/AA (not provided)
Unit dimensions and weight.....	69 x 43 x 87 (h) mm - 106 g
Software version.....	BCS - B
Automatic stop at end of each program.....	Yes - about 4 min 30 sec
Activation safety.....	100%
Detection absence of electrodes or of contact.....	yes detection independent on each of the 2 outputs
Indicator of power.....	bar graph - LCD display
Indicator of weak and/or defective battery.....	yes - LCD display
Indicator absence contact belt.....	yes - LCD display
Indicator of the programs.....	LCD display : prog 1-2-3-4
Temperature of storage / Relative humidity of storage.....	0°C to + 45°C / 10% to 90%
Temperature of use / Relative humidity of use.....	+ 5°C to + 45°C / 20% to 65%
Connection device - belt.....	wires - plugs ø 2 mm male
Caution.....	⚠ Read the manuel thoroughly
Guarantee / Lifecycle.....	2 years

Device in contact with the patient, excluding the heart muscles.

**RoHS (2002/95/EC)**..... The unit are free of materials likely to have a negative impact on the environment.

**IP 22**..... Protected against the ingress of foreign object ≥12.5 mm & against dripping water when tilted up to 15°.

### BELT & PANTY:

Size (unisex).....	from 34 to 60
Length of the unfolded belt.....	49,6 inch/126 cm
Fabric.....	polyester
Built-in electrodes composition.....	carbonated silicon.
Belt pads surface.....	4 x 28 cm <sup>2</sup>
Panty pads surface.....	4 x 25 cm <sup>2</sup>
Class I medical device in accordance with Directive 93/42/CEE	

### CONTACT CREAM

Packaging.....	75 ml / 2,54 Fl Oz
Manufacturer's reference.....	VC 57B/53-148
Class I medical device in accordance with Directive 93/42/CEE	

### GUARANTEE :

- ▶ The belt, the panty and the cream are designed for personal use. For reasons of hygiene, there will be neither trade-in nor exchange. The belt and panty life is limited to 12 months.
- ▶ Do not, under any circumstances, attempt to open or repair the machine yourself! Only have repairs carried out by the customer service department. If these instructions are not heeded, the warranty becomes null and void.
- ▶ Use this device only with the accessories recommended by SPORT-ELEC®.

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## GB - WARRANTY

This appliance is guaranteed for 24 months from the date of purchase. During the period of the guarantee, the distributor will take responsibility for defects occur, the appliance will be repaired or exchanged as the distributor deems necessary. Damage arising from improper use or normal wear and defects, which do not materially affect the performance of the appliance, is not covered under this guarantee. The guarantee is not valid if unauthorized repairs have been attempted or if replacement parts used are not approved by the supplier. The guarantee is valid only if it is stamped and signed by the retailer at the time of the purchase. For all claims made during the period of the guarantee, send the appliance and the warranty card to your retailer. All other claims are excluded from the guarantee unless specifically prohibited by law.

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